EN - SLIP-ON PADS

Installation & User Instructions

SP - ALMOHADILLAS DESLIZABLES

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DE - AUFSCHIEBBARE POLSTER

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- P-PBPADS
- P-PBPADM
- P-PBPADL

Thank you for choosing this **POZIFORM® SLIP-ON PADS**

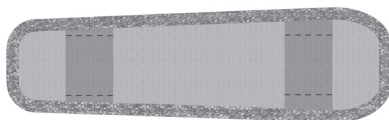
Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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EN - ASSEMBLY INSTRUCTIONS

BEFORE fitting the slip-on pads to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

1. Slide the strap under both of the loops as shown.
2. Slide the Comfort Pad into required position.

EN - CLEANING & MAINTENANCE

Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.

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EN

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate

11724 Willake Street | Santa Fe Springs | CA 90670
T. 800-454-6612 | P. 562-282-0244 | F. 310-305-1718
www.inspiredbydrive.com | info@inspiredbydrive.com