



Ultima™ Access Bath Chair

U. S. Patent No. 7,328,953; D503,556; D507,904, D508,175

BCUA-8626M

BCUA-8726L

BCUA-8926XL

www.inspiredbydrive.com



WARNING!

Read ALL instructions before using this product!

ATTENTION

Your Ultima Access Stainless Steel Bath Chair has been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety information provided.

Here are some additional precautions you can take to prevent the risk of injury:

1. Read and understand ALL instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
- 3. To reduce the risk of injury or drowning, NEVER leave the user unsupervised or unattended while using this product.**
4. Check regularly that all tubing, bolts, buckles, and other parts are tight at ALL times.
5. When occupied, make sure the user is buckled in at all times.
6. Chair legs should NEVER be in complete vertical positions.
7. SAVE THESE INSTRUCTIONS.

If you require additional assistance, please contact Inspired by Drive Customer Service at (800) 454-6612, or email info@inspiredbydrive.com.

Thank you for choosing Inspired by Drive.

ACCESSORIES

To order accessories for your Ultima Access bath chair, please contact customer service at **(800) 454-6612**, or send an email to info@inspiredbydrive.com.

ACCESSORIES




Pelvic Positioning Belt



Detachable Extension Base

Raise height by 9.5" (241.3 mm). Max. weight capacity 180 lbs (82 kg). Ideal for tubs.

ASSEMBLY OF CHAIR

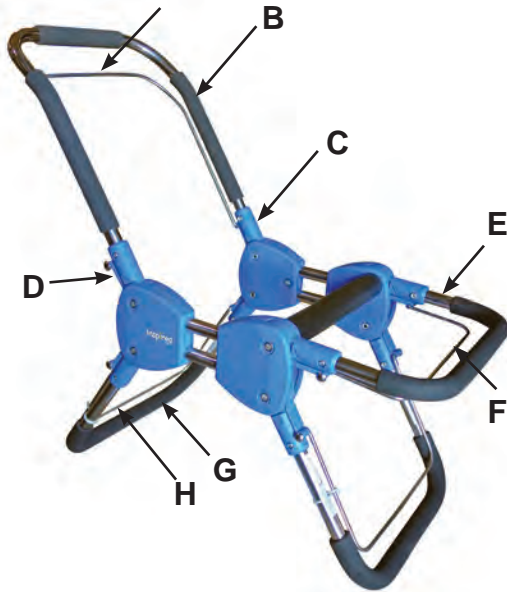

- A. Back angle adjustment pull handle
- B. U-shaped back support frame
- C. Insertion point for back frame
- D. Back angle adjustment pull tab
- E. U-shaped calf support frame
- F. Calf angle adjustment pull handle (straight calf panel)
- G. Leg frame (can be positioned at 0°, 30°, 60° angles, relative to seat's horizontal plane)
- H. Leg frame adjustment pull handle

Assembling the Back Support:

1. One by one, pull on each of the pull tabs and change the pivot angle to vertical (Fig. 1). Insert the U-shaped stainless steel back and calf frames into the pivots and fasten with bolts and nuts provided.
2. Insert the ends of the angle adjustment pull handles into the small holes located on the pull tab on each side of the frame (Fig. 2).

Firmly press the stainless steel caps over the ends of the pull handle. Please ensure that the end cap fits tightly over the groove on the pull handle. Test it by tugging at the cap. If it comes off, then reinstall.

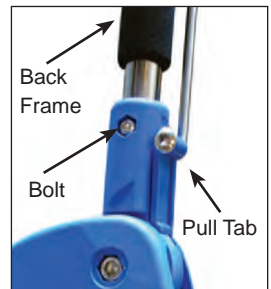




Figure 1



Figure 2

PROPER ASSEMBLY CHECK

Before each use of your bath chair, please verify that the back and calf supports are properly installed and locked into position. When correctly installed, the pull tab should click into place and completely engaged into the slot. There should not be a gap between the pull tab and connector. See the figures below.

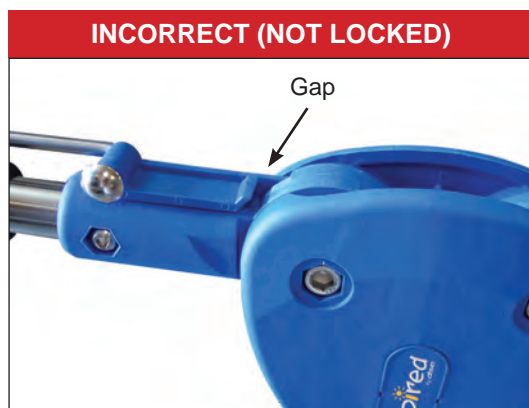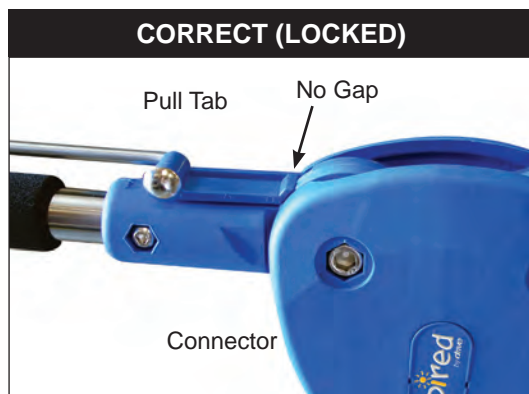

ATTENTION

Adjustments to the Ultima™ Access™ SHOULD NOT be made while the product is in use or occupied.

COVER INSTALLATION AND ANGLE SETTINGS

Cover Installation: (refer to chair diagram on page 4)

- Pull on the angle adjustment pull handles (A, F) to set the backrest and calf supports to the most upright position (Fig. 3). Slip the cover over the backrest and adjustment pull handle and then calf support and pull handle (Fig. 4). Wrap the hook & loop straps around the top side-tube and secure underneath the chair (Fig. 5).



Figure 3

Adjusting Back Angle:

1. Locate the adjustment pull handle (A) on the back of the chair under the cover (it is not necessary to remove the cover to operate the lever). Using both hands, grip the lever and back frame and squeeze together.

2. With a lifting motion, adjust the back angle up or down.

NOTE: Back angle should not be adjusted when the chair is occupied.



Figure 4

Adjusting Calf Support Angle:

1. On the calf support frame, under the cover, locate the adjustment pull handle (F). It is not necessary to remove the cover. Using both hands, grip the pull handle and calf support frame and squeeze together.

2. With a lifting motion, adjust the angle up or down.

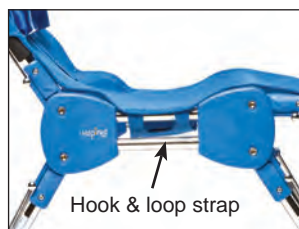


Figure 5

Note: The legs can be positioned at 0° (horizontal), 30°, and 60° angles (relative to the seat's horizontal plane).

⚠ WARNING

The legs **SHOULD NEVER** be in a complete 90° angle (perpendicular to the seat's horizontal plane). Leg angles greater than 60° will cause the chair to be unstable.

SEAT INSTALLATION

Push the rear of seat cushion on the back support frame, pull the front of the cushion forward and push down to snap on the front frame.

CLEANING INSTRUCTIONS

Soapy water or mild cleansers may be used on all parts of your Ultima Access Bath Chair. Harsh abrasive cleansers may dull the finish.

FABRIC CARE

The fabric cover for Ultima Access Bath Chair is made of polyester that is durable, antimicrobial, resilient, quick drying, fade resistant, and machine washable.

Cleaning Instructions:

1. Remove the chair cover by following these steps below:
 - Adjust the back and calf support frames to horizontal. Turn the chair up-side down and remove the hook & loop strap.
 - Slip the cover over the back and calf supports.
2. Using warm water (100°F/38°C) and household laundry detergent, wash the cover in the washing machine using gentle cycle.
3. Air dry.

MAXIMUM WEIGHT CAPACITY

To ensure the occupant's safety, DO NOT exceed the following maximum weight limits:

| MODEL NO. | MAXIMUM WEIGHT CAPACITY |
|-------------|-------------------------|
| BCUA-8626M | 130 lb. (59 kg) |
| BCUA-8726L | 160 lb. (73 kg) |
| BCUA-8926XL | 180 lb. (82 kg) |

WARRANTY

Inspired by Drive warrants this product to be free of defects in material and workmanship. Our obligation under this warranty is limited to repair or replacement of any part or entire unit at our option for a period of one year from date of delivery to the original purchaser. The warranty does not include cost of inconvenience, property damage, misuse, abuse, crash or similar incidents. The warranty will not apply if product has been tampered with or repaired by unauthorized individuals.



© 2017 Columbia Medical, LLC. dba Inspired by Drive
11724 Willake Street | Santa Fe Springs, CA 90670
Toll Free: 800-454-6612 | Direct: 562-282-0244
www.inspiredbydrive.com | info@inspiredbydrive.com