
One Year Limited Warranty

Your Inspired by Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.



© 2016 Inspired by Drive. All rights reserved.

Inspired by Drive is a trademark of Medical Depot, Inc.
Santa Fe Springs, CA 90670 USA

Rev2.11.01.16



Seat2Go

Item # CRS 2000 & CRS 3000



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.inspiredbydrive.com

assembly


Your Seat2Go comes in one carton completely assembled.

use & adjustments

Your Seat2Go can be used as a floor sitter, booster chair and as an insert in a stroller.

floor sitter


1. Place Seat2Go on a flat surface.
2. Open Fastex™ clip (A) on pelvic belt. Detach two shoulder straps (B) from hook and loop fastener strips on Seat2Go back. Open Fastex™ clip (C) on chest support (D) and pull support apart.
3. Place child in Seat2Go and fasten pelvic belt.
4. Wrap chest support (D) snugly around child, attach hook and loop fastener and fasten Fastex™ clip.
5. Determine correct fit for shoulder straps (B) and attach to Hook and loop fastener strips on Seat2Go back.

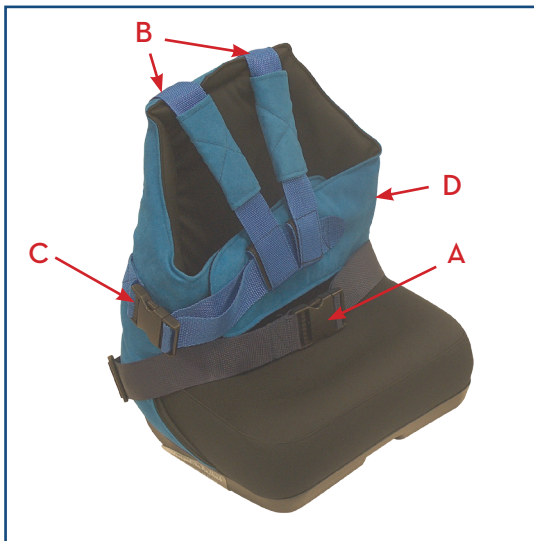

booster chair and stroller



1. Remove fastening belts from pocket on seat bottom (E) and Seat2Go back (F).

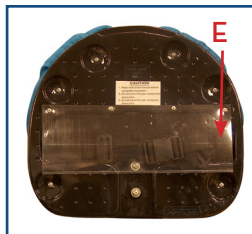

2. Place Seat2Go on any chair or in a stroller, wrap belts around the back and seat of chair/stroller and close the buckles. NOTE: Make sure belts are taut.
3. Insert child as per instructions for floor sitter.

accessories

(sold separately)

CRS 8000
CRS 8025

Headrest
Abductor