EN - ANKLE POSITIONERS

Installation & User Instructions

SP - POSICIONADORES DE TOBILLOS

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Thank you for choosing this **POZIFORM® ANKLE POSITIONERS**

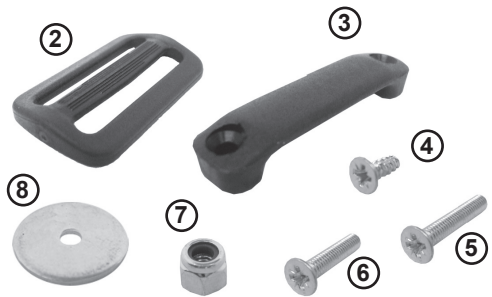
Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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CONTENTS / CONTENIDO / CONTENU / INHALT

EN

- | | |
|---------------------------|--------------------------|
| 1. Ankle Positioner x 2 | 5. Screw (1" long) x 8 |
| 2. Three-Bar Slide x 4 | 6. Screw (3/4" long) x 8 |
| 3. Webbing Bridge x 4 | 7. Lock Nut x 8 |
| 4. Self Tapping Screw x 8 | 8. Washer x 8 |

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EN - ASSEMBLY INSTRUCTIONS

BEFORE fitting the ankle positioners to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

Ankle Positioner Assembly

1. Place the user's foot or shoe on the footplate and set the webbing bridge clear of the foot (Fig. 1), in line with the ankle bone. Mark the holes to be drilled.
2. Drill 4 holes in each foot plate to suit either assembly method (Fig. 2 & Fig. 3).
3. Attach the webbing bridge with the provided hardware.
4. Insert the three-bar slide on the ankle holder strap and thread the strap under the webbing bridge and back into the slide (Fig. 5). **Note:** Do not tighten the strap at this point.

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

EN - ASSEMBLY INSTRUCTIONS

Ankle Positioner Adjustment

5. Place the positioner above the ankle, aligning the outer pads with the ankle bones and the rear of the positioner just above the heel, then feed the strap through the buckle (Fig. 6).
6. Pull the strap through the buckle until a tight/comfortable fit is achieved (Fig. 7). Press the hook and loop together.
7. Hold onto the three-bar slide and pull the strap tight until the positioner holds the foot firmly to the footplate (Fig. 8). Thread the strap back through the three-bar slide (Fig. 9).
8. Follow the above steps for the other positioner.
9. To release the positioner, follow the above steps in reverse. Do Not adjust the webbing bridge straps unless you wish to alter position.

Figure 6

Figure 7

Figure 8

Figure 9

Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate