EN - CALF PANEL/CALF STRAP

Installation & User Instructions

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DE - WADENPLATTE/WADENBAND

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- P-CS180S
- P-CS180M
- P-CS180L
- P-CP280S
- P-CP280M
- P-CP280L

The **POZIFORM®** Calf Panel/Calf Strap provides a lightweight, padded support that prevents the wheelchair users' feet from sliding off the back of the footrests of the wheelchair.

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EN

- Padded Calf Panel or Padded Calf Strap

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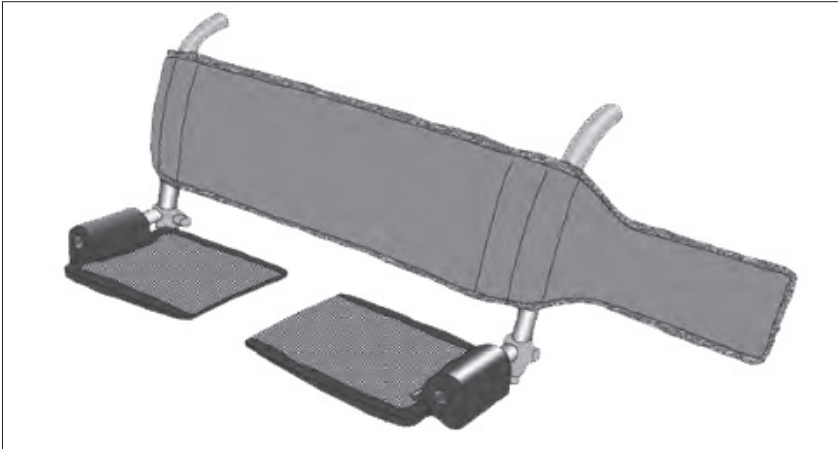
EN - ASSEMBLY INSTRUCTIONS


BEFORE fitting the calf strap/calf panel to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

1. Wrap the Calf Panel/Calf Strap around the front of the footrest hangers as shown.

EN - ASSEMBLY INSTRUCTIONS

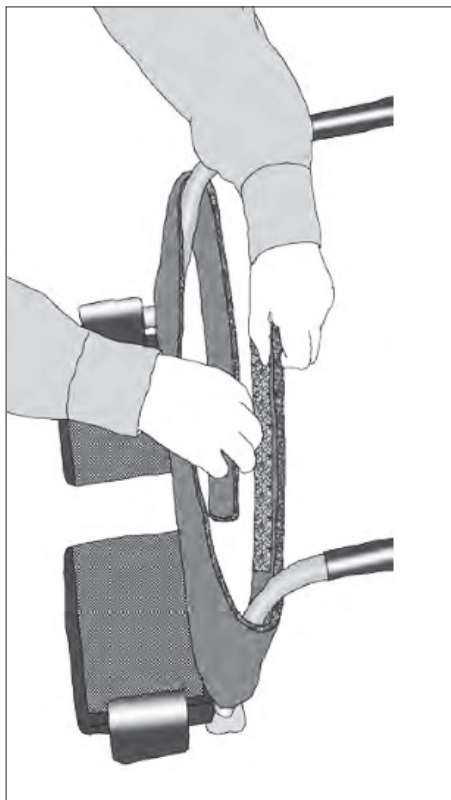2. Pull the Calf Panel/Calf Strap tight and press the two sides of the hook and loop together.

NOTE: You must remove the Calf Panel/Calf Strap before removing or swinging the footrest aside to prevent damage to the Calf Panel/Calf Strap and wheelchair.

Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate