**EN - ELASTIC STRAPS**

Installation & User Instructions

**SP - CINTA ELÁSTICA**

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**DE - ELASTISCHER GURT**

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- P-DCSS
- P-DCSL
- P-DCSXL

Thank you for choosing this **POZIFORM® ELASTIC STRAP**

Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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- Elastischer Gurt x 1

## EN - ASSEMBLY INSTRUCTIONS

**Poziform Elastic Straps can be used in a variety of ways to provide moderate support on a short term basis.**

1. Elastic straps can be wrapped around the trunk and the backrest of the wheelchair to provide trunk support.

2. Elastic straps can also be used for lower extremity positioning. They can provide adduction of the lower extremities when wrapped around the thighs. They can also be used to provide temporary positioning lower on the legs near the calves or ankles.

3. Elastic straps can be shortened and used to temporarily position upper extremities also.

## EN - ASSEMBLY INSTRUCTIONS

### Precautions:

1. Elastic straps should not be wrapped too tightly around any part of the body.
2. Check for adequate clearance by inserted 2 fingers between the elastic strap and the end user.
3. Do not use elastic straps for positioning of the head or near the neck.
4. Skin checks should be completed every 20 minutes while using an elastic strap.
5. Discontinue use of the elastic strap if any irritation or redness is present.

### Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

### Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate