EN - FOOT STRAPS

Installation & User Instructions

SP - CINTAS PARA PIES

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- P-TS100
- P-TS200

'D'
RING

Thank you for choosing this **POZIFORM® FOOT STRAPS**

Foot Straps are sold in pairs which can be used on one foot (as shown) or individually.

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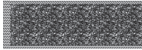
②

EN

1. Foot Strap x 2
2. Self Adhesive Hook and Loop Pad x 2

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EN - ASSEMBLY INSTRUCTIONS

BEFORE fitting the foot straps to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

Foot Strap Assembly

If your footplates have slots in them, thread the foot straps through the slots (Fig. 1). Alternatively, you can drill through the straps and attach with screws/rivets etc. (not supplied) to the footplate (Fig. 2).

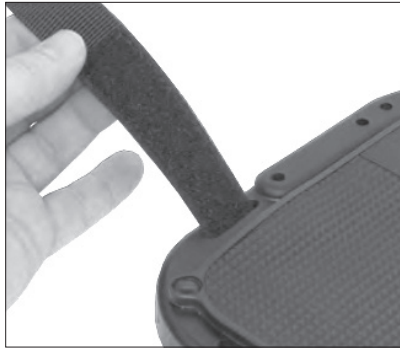
Figure 1

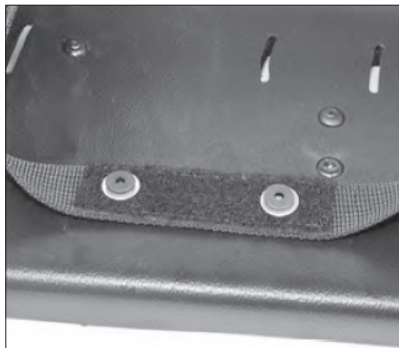
Figure 2

EN - ASSEMBLY INSTRUCTIONS

Foot Strap Adjustment

Standard Foot Strap

- Grab both halves of the strap, pull until snug (Fig. 3) and press the two halves of hook and loop together.

Figure 3

'D' Ring Foot Strap

- Hold on to the 'D' ring, pull the strap through until snug (Fig. 4) and press the two halves of hook and loop together.

Figure 4

Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate