**EN - REAR PULL/NARROW CONTOURED CHEST HARNESS &  
REAR PULL BACKPACK STYLE CHEST HARNESS**  
Installation & User Instructions

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Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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## CONTENTS / CONTENIDO / CONTENU / INHALT

EN

- |  |   |
|--|---|
| <ol style="list-style-type: none"> <li>1. Contoured or Rear Pull Backpack Style Chest Harness x 1</li> <li>2. Lower Strap x 2</li> </ol> | <ol style="list-style-type: none"> <li>3. Upper Strap with adjuster and side release buckle x 2</li> <li>4. Cam Buckle x 4</li> <li>5. Three-Bar Slide x 2</li> </ol> |
|--|---|

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## EN - ASSEMBLY INSTRUCTIONS

**BEFORE** fitting the chest belt to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

### Standard Solid Backrest Installation with Rear Pull Adjusters

The following fitting instructions are only guidelines as seating systems vary according to manufacturer.

1. Identify mounting location for all upper and lower straps on the backrest. Drill holes or locate T-nut positions on the backrest.
2. Insert Upper Straps through Three-Bar Slides and loop through side release buckles (Fig. 1). Finish by routing back through Three-Bar Slides (Fig. 2).

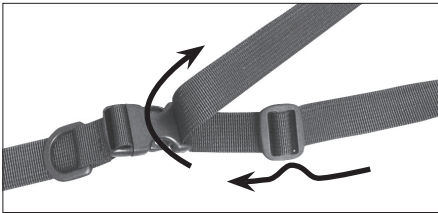
Figure 1

Figure 2

3. Attach Upper Straps to the backrest with appropriate screws or bolts (not included) as shown in Figure 3.
4. Attach Lower Cam Buckles to backrest with appropriate screws or bolts (not included), see Fig. 4.
5. Thread the Lower Straps through Lower Cam Buckles.
6. Place the harness over the user's head and shoulders. With the Lower Cam Buckles open, pull the straps for a secure fit to the user and close the cam buckles to in position. Tighten the top straps with 'D' Ring Adjusters.

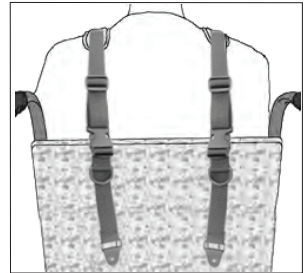
Figure 3

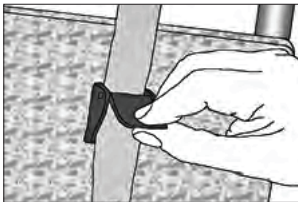
Figure 4

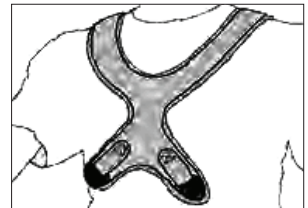
Figure 5

## EN - ASSEMBLY INSTRUCTIONS

### Alternate Solid Backrest Installation with Cam Buckles (Direct Mounting)

The following fitting instructions are only guidelines as seating systems vary according to manufacturer.

1. Identify mounting location for all four cam buckles on the backrest. Drill holes or locate T-nut positions and fasten the cam buckles with appropriate screws or bolts (not included). See Fig 6 & 7.

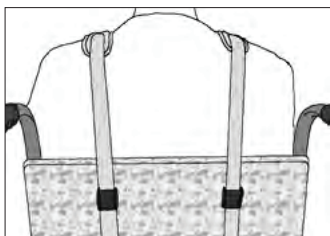
Figure 6

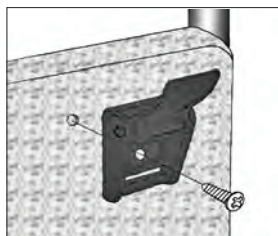
Figure 7

2. Thread upper straps through upper cam buckles and lower straps through lower cam buckles (Fig. 8).

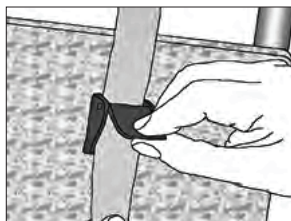
Figure 8

3. Place the harness over the user's head and shoulders. With the cam buckle open, pull the strap for a secure fit to the occupant's and close the cam buckle to lock in position. Repeat for all four straps.

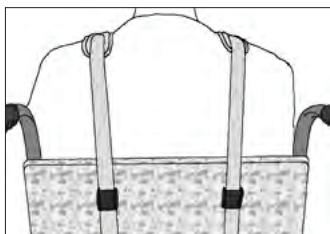
Figure 9

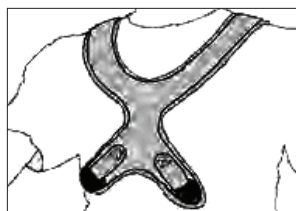
Figure 10

### Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

### Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate