**EN - HOOK & LOOP CHEST BELT**

Installation & User Instructions

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- P-CB700
- P-CB800P

Thank you for choosing this **POZIFORM® PELVIC BELT**

Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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EN

1. Chest Belt x 1
2. Strap Guide x 2
3. Three-Bar Slide x 2

SP

1. Chest Belt x 1
2. Strap Guide x 2
3. Three-Bar Slide x 2

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## EN - ASSEMBLY INSTRUCTIONS

**BEFORE** fitting the chest belt to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

### Chest Strap Assembly

1. Determine the appropriate location for the mounting hardware to be installed on the wheelchair.
2. Attach the Strap Guides to the wheelchair by fastening screw (not provided) to the frame (Fig. 1). Note: the Strap Guide can be mounted either on the back or on the side of the wheelchair back.
3. Slide one Three-Bar Slides onto each end of the Chest Belt (Fig 2).
4. Thread both Chest Belts through the Strap Guides (Fig. 3) making sure the 'D' Ring is on top.
5. Thread each end the second Chest Belt back through the Three Bar Slide (Fig. 4).
6. Thread the front end of the Chest Belt through the 'D' Ring and fasten where desired.

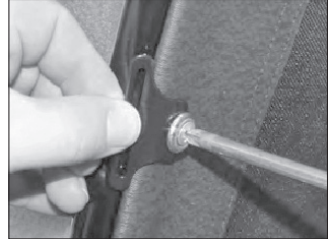
Figure 1

Figure 2

Figure 3

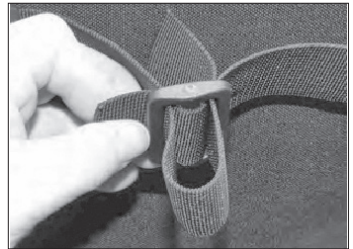
Figure 4

Figure 5

### Chest Strap Adjustment

7. Adjust the rear attachment of the Chest Belt to the desired tension until the belt is not too loose to be ineffective nor too tight to cause discomfort.

### Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

### Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate