EN - POZIFLEX™ CHEST STRAP

Installation & User Instructions

SP - CINTA PARA PECHO POZIFLEX

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DE - POZIFLEX-BRUSTGURT

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- P-CB900XS
- P-CB900S
- P-CB900M
- P-CB900L
- P-CB900XL

Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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EN - ASSEMBLY INSTRUCTIONS

BEFORE fitting the chest belt to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

Chest Strap Assembly

1. Determine the appropriate location for the mounting hardware to be installed on the wheelchair.
2. Attach the Strap Guides to the wheelchair by fastening screw (not provided) to the frame (Fig. 1). Note: the Strap Guide can be mounted either on the back or on the side of the wheelchair back.
3. Thread both Chest Straps through the Strap Guides (Fig. 2) making sure the 'D' Ring is on top.
4. Slide both Three-Bar Slides onto the Chest Strap (Fig 3).

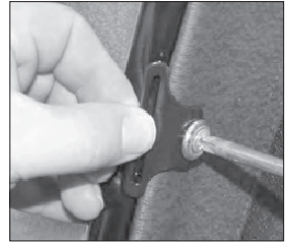
Figure 1

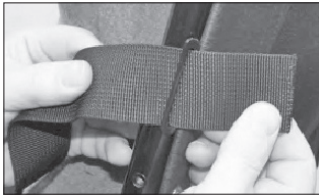
Figure 2

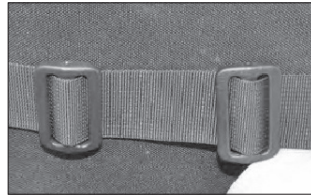
Figure 3

5. Thread the second Chest Strap through the first Three Bar Slide (Fig. 4 & Fig. 5).

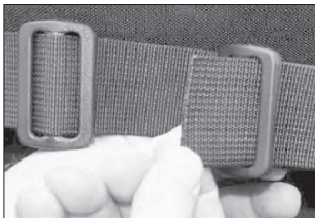
Figure 4

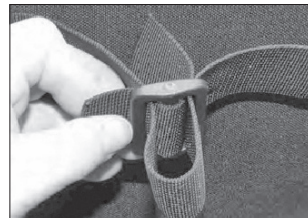
Figure 5

6. Thread the second Chest Strap through one side of the second Three-Bar Slide (Fig. 6).
7. With the user in place, position the Chest Pad just below the chest and clip the buckles (Fig. 7).

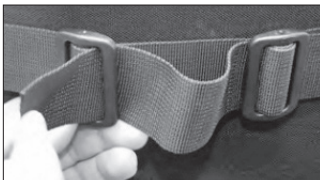
Figure 6

Figure 7

EN - ASSEMBLY INSTRUCTIONS

Chest Strap Adjustment

1. With the user in the wheelchair, pull the Chest Strap so that there is only 3" (75mm) of strap from the buckle to the 'D' ring (Fig. 8).
2. Adjust the slack on the strap by pulling through the Three Bar Slides, making sure the slides are evenly spaced, roughly in the middle of the wheelchair back. Thread the second Chest Strap fully through the second Three Bar Slide (Fig. 9) to lock off the two Chest Straps.

Figure 8

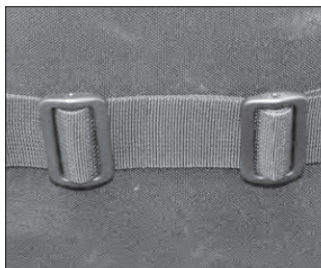
Figure 9

3. Tighten the Chest Strap by holding on the buckle and pull the 'D' Ring (Fig. 8) until the strap is not too loose to be ineffective nor too tight to cause discomfort.
4. To release the tension on the strap, pull the buckle (Fig. 10) and pull the strap through.
5. To release the Chest Strap, press the buckles (Fig. 11) and remove the Chest Pad.

Figure 10

Figure 11

Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate